

DPNF Bedpan - European bedpan with handle and lid

Long Life Re-usable Polypropylene Bedpan for use in Nursing, Rehabilitation and Elderly Care



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



Warwick House, Heathcote Way, Warwick, CV34 6TE T: 00 44 (0)1926 422427 www.sasco.co.uk

Product Benefits

- Suitable for both male and female use.
- Ideal for patients confined to bed.
- Lightweight, strong and durable.
- Secure fitting lid to prevent leaks and spills and reduces unpleasant odours and risk of cross infection.
- Can be cleaned in automatic washer disinfectors or hand washed.
- Smooth material surfaces ensure ease of use and ease of cleaning.

Product Specification

- European style circular bedpan with handle and lid
- 420mm x 315mm x 100mm
- Pack Size 15
- Pack Weight 8 kg
- The following standards are applicable:

ISO 9001:2008

- **Quality Management System**
 - ISO 13485:2003
 - Medical Devices Quality Management System
- BS2588: 1992 **Reusable Portable Sanitary Pans**
- Manufactured from polypropylene
- Sterilizable up to 134°C

Guide to Using Spa Products from Warwick SASCo

Usage

- Suitable for male and female use.
- For use in a seated position.
- Close lid after use to reduce odour and risk of spills.
- Empty after each use.

Cleaning

- For Hospital Use Wash before first use. Clean in automatic washer disinfectors compliant to ISO 15883.
- Wash cycle at 80°C. Disinfection cycle at 90°C. For further processing information, refer to machine manufacturers instructions and local hospital policy.
- For Personal Use Wash before first use in hot water with detergent.
 - Empty contents after use down a toilet. Rinse pan empty down a toilet.
- Wash in hot water with detergent after each use and allow to dry naturally.

Storage

This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to. Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.