

CBL Commode Pan - Commode pan with lid

Long Life Re-usable Commode Bucket compatible with Commode Chairs for use in Nursing, Rehabilitation, Elderly Care and Community Care



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.

Product Benefits

- Lightweight, strong and durable.
- For use with British Standard commode chairs
- Secure fitting lid to prevent leaks and spills and reduces cross infection and unpleasant odours.
- Smooth material surfaces ensure ease of use and ease of cleaning.

Product Specification

- Commode Bucket with Lid
- 259mm x 259mm x 144mm
- Pack Size 25
- Pack Weight 10kg
- The following standards are applicable:
 - ISO 9001:2008
 - Quality Management System
 - ISO 13485:2003
 - Medical Devices Quality Management System
 - BS2588:1995
 Reusable Portable Sanitary Pans
- Manufactured from Polypropylene
- Maximum temperature 100°C

Guide to Using Spa Products from Warwick SASCo

Usage

- For male and female use.
- For use in a seated position with a commode chair.
- Close lid after use to reduce odour and risk of spills.
- Empty after ease use.

Cleaning

- Product should be washed before first use. Wash in hot water with detergents and allow to dry naturally.
- After use wash in hot water using detergents and disinfectants.
 - Spa Products are designed for a repeated use guaranteed for 1000 washes.
- Scouring and abrasive cleaners will reduce the performance of products.

Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
 - Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.

