

# Spa Range

## PSPS Bedpan Support - For pulp disposable midi slipper pan

Long Life Re-usable Polypropylene Slipper Pan Support for use in Nursing, Rehabilitation, Elderly Care



### *Incorporates Biomaster Protection*

*A safe, effective and permanent antimicrobial additive based on silver and proven to reduce the growth of harmful bacteria by up to 99.9%*

The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



### Product Benefits

- Use to support pulp bedpan products to improve durability and hygienic handling of pulp bedpan products.
- Ideal for patients confined to bed due to illness or after surgery.
- Lightweight, strong and durable.
- Protected with Biomaster Antimicrobial Additive.
- Smooth material surfaces ensure ease of use and ease of cleaning.
- Can be cleaned in automatic washer disinfectors or hand washed.

### Product Specification

- Adult Bedpan Support for use with Pulp Bedpan Products - 1.5 Litre
- 470mm x 290mm x 75mm
- Pack Size 30
- Pack Weight 7 kg
- The following standards are applicable:
  - **ISO 9001:2008**  
Quality Management System
  - **ISO 13485:2003**  
Medical Devices - Quality Management System
  - **EU Directive 528:2012 Biocide**  
Availability on the Market and the Use of Biocidal Products
- Manufactured from Polypropylene

### Guide to Using Spa Products from Warwick SASCo

#### Usage

- Suitable for male and female use.
- For use in a seated position.
- Empty after each use.

#### Cleaning

- **For Hospital Use** - Wash before first use. Clean in automatic washer disinfectors compliant to ISO 15883.
- Wash cycle at 80°C. Disinfection cycle at 90°C. For further processing information, refer to machine manufacturers instructions and local hospital policy.
- **For Personal Use** - Wash before first use in hot water with detergent.
- Empty contents after use down a toilet. Rinse pan empty down a toilet.
- Wash in hot water with detergent after each use and allow to dry naturally.

#### Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.