Spa Range



UF1000 Urinal Bottle - Female urinal with handle

Long Life Re-usable Plastic Female Urinal Bottle for use in Nursing, Rehabilitation, Elderly Care and Personal Home Use



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



Product Benefits

- Anatomically shaped for female use.
- Lightweight, strong and durable.
- Clear, easy to read graduations.
- Easy grip handle particularly suitable for those with limited hand movement.
- Smooth material surfaces ensure ease of use and ease of cleaning.
- Guaranteed for a minimum of 1000 wash cycles.

Product Specification

- Female Urinal Bottle Graduated, 1000ml
- Pack Size 30
- Pack Weight 5.8kg
- The following standards are applicable:
 - ISO 9001:2008

Quality Management System

- ISO 13485:2003

Medical Devices - Quality Management System

- BS3215:1995

Re-usable Portable Urinals

Manufactured from Polyethylene (HDPE)

Guide to Using Spa Products from Warwick SASCo

Usage

- For female use only.
- Can be used in seated, standing or supine positions.
- Empty after each use.

Cleaning

- Product should be washed before first use. Wash in hot water with detergents and disinfectants.
- Maximum water temperature 90°C.
- After each use, wash in hot water with detergents and disinfectants and allow to dry.
- Scouring and abrasive cleaners will reduce the performance of products.

Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.