

# Spa Range

## UMT Non Leak Valve - For use with UMA1000 & UML1000 Contains Latex

Long Life Re-usable Plastic Non Return Valve for use with Male Urinal Bottle in Nursing, Rehabilitation, Elderly Care, Community Care and Personal Home Use



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.

### Product Benefits

- Non return valve for use with UML1000 Adult Male Urinal Bottle 1000
- Secure fitting valve prevents leaks and spills and reduces unpleasant odours and risk of cross infection.
- Smooth material surfaces ensure ease of use and ease of cleaning
- Ensures one-way only liquid flow when using urinal bottle.
- Replacement spare parts available to order separately.

### Product Specification

- Non Return Valve for use with UML1000 Male Urinal Bottle
- Pack Size 25
- The following standards are applicable:
  - **ISO 9001:2008**  
Quality Management System
  - **ISO 13485:2003**  
Medical Devices - Quality Management System
  - **BS3215:1995**  
Re-usable Portable Urinals
- Valve body manufactured from Polypropylene
- Valve collar and teat manufactured from Latex

### Guide to Using Spa Products from Warwick SASCo

#### Usage

- For with UML1000 Male Urinal Bottle 1000ml only.
- For use in a seated, standing or supine position.
- Empty urinal bottle after each use.

#### Cleaning

- **For Hospital Use** - Wash before first use.
- Maximum washing temperature for non return valve 85 °C. For further processing information, refer to machine manufacturers instructions and local hospital policy.
- Check latex teat is functioning by gently rolling between fingers before use.
- **For Personal Use** - Wash before first use in hot water with detergent.
- Check latex teat is functioning by gently rolling between fingers before use.
- Empty urinal bottle contents immediately after use.
- Wash in hot water with detergent after each use and allow to dry. Scouring and abrasive cleaners will reduce the performance of the product.

#### Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.

