

Spa Range

USP1200 Urinal Bottle with UMT - Male urinal bottle with non leak valve Contains Latex

Long Life Re-usable Plastic Male Urinal Bottle for use in Nursing, Rehabilitation,



The Spa Range is a range of re-usable living aids designed for use in Hospital Wards, Nursing and Care Homes, Rehabilitation and Personal Use at home.

Our Spa Range is an attractive, stylish and functional group of products which includes drinking cups, beakers, jugs, vomit bowls, medicine cups, urinals, bedpans and commodes.



Product Benefits

- Lightweight, strong and durable male urinal bottle
- Secure fitting cap to prevent leaks and spills and reduces unpleasant odours and risk of cross infection
- Graduated with clear, easy to read measurements
- Easy grip handle - particularly suitable for those with limited hand movement
- Hollow handle for improved cleaning results
- Smooth material surfaces ensure ease of use and ease of cleaning
- Valve prevents leaks and spills and ensures one-way only liquid flow

Product Specification

- Male Urinal Bottle - Graduated, 1000ml with Non Return Valve
- Pack Size 10
- The following standards are applicable:
 - **ISO 9001:2008**
Quality Management System
 - **ISO 13485:2003**
Medical Devices - Quality Management System
 - **BS3215:1995**
Re-usable Portable Urinals
- Urinal bottle manufactured from Polyethylene (HDPE)
- Valve body manufactured from Polypropylene
- Valve collar and teat manufactured from Latex

Guide to Using Spa Products from Warwick SASCo

Usage

- For male use only.
- For use in a seated, standing or supine position.
- Close cap immediately after use to prevent spillage and odours.
- Empty after each use.

Cleaning

- **For Hospital Use** - Wash before first use.
- Maximum washing temperature for urinal bottle 90°C
Maximum washing temperature for non return valve 85°C. For further processing information, refer to machine manufacturers instructions and local hospital policy.
- **For Personal Use** - Wash before first use in hot water with detergent.
- Empty contents immediately after use.
- Wash in hot water with detergent after each use and allow to dry. Scouring and abrasive cleaners will reduce the performance of the product.

Storage

- This product has a shelf life of 36 months from date of manufacture providing the recommended storage conditions are adhered to.
- Product should be stored in its primary packaging, off the floor on a pallet or shelf, and in a clean dry ventilated room at a temperature of 4 °C - 25 °C.