


Product Profile

UMT Non leak value for UMA1000 & UML1000

Dimensions : N/A
 Volume : N/A
 Material : Polypropylene
 Colour : White
 Pack Size : 25

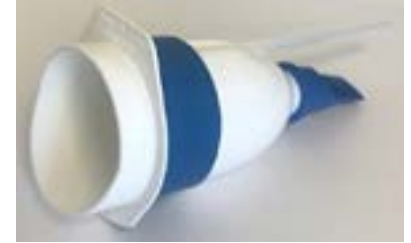

Intended Use

- Suitable for use with the UMA1000 and the UML1000
- Bedpan for use in Nursing, Rehabilitation, Elderly care and Personal home use
- Reusable product

Product Features

- Secure fitting valve prevents leaks and spills and reduces unpleasant odours and risk of cross infection
- Ensures one-way only liquid flow when using urinal bottle
- Lightweight, strong and durable
- Contains latex
- Contains instruction leaflet
- Guaranteed 1000 cycles

Packaging Information

Product Code	Case Bar Code (GTIN 14)	Primary Packaging	Secondary Packaging	Case Weight
UMT	15060098135413	Individually bagged	25 in cardboard box	2 kgs

Product Symbols



Additional Information & Standards

- Not a measuring device
- Tariff Code 39.26.9097 Hospital Medical Holloware
- Country of origin United Kingdom
- Product compatible with BS EN ISO 13485 Medical Device - Quality Management System
- Company Compliant with BS EN ISO 2588 Portable sanitary pan for use in healthcare establishments
- Additional document available on request contact enquiries@sasco.co.uk



Instructions For Use

Storage and handling

- Do not drop
- Store in original packaging
- Store in clean and dry conditions
- Store off the floor on a shelf or pallet
- Shelf life 36 months

Preparation

- Inspect products before processing
- Withdraw damaged products and report to manager
- Rinse with warm water to remove any packing residues
- No particular requirements

Cleaning and Disinfection

- Wash before use
- Machine clean in Washer Disinfectors compliant with ISO15883
- Wash cycle at 80°C. Disinfection cycle at 90°C
- Manual (hand) washing is permissible

Inspection and Maintenance

- Visually inspect product to identify damage
- Discard damaged product

Disposal

- End of life is determined by wear and tear, visual inspection or damage to product
- If product is contaminated treat as clinical waste and dispose of in accordance with clinical waste procedures

Warnings

- Do not use abrasive powders or metal brushes as they may cause surface damage
- Contains latex

Important Information For Processing

- The information provided by Warwick SASCo is the company's official instructions for this product
- It is the responsibility of processor to ensure the processing is performed in accordance with this information and that personal, machines and equipment comply with relevant standards for processing
- Any deviation from this information requires evaluation for the effectiveness and potential adverse consequences

